E v o l u t i s
C R E A T E U R F A B R I C A N T

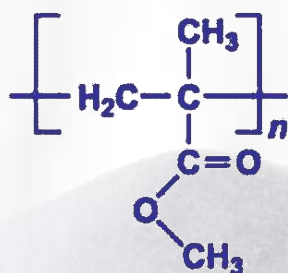
EvoCem®

1

G1

3

G3

Bone Cement

*Ciment de fixation
osseuse*

1 EvoCem®

Acrylic bone cement are indicated for immediate and long term fixation of metal and polyethylene orthopaedic implants destined for use with cement.

Radiopaque and self curing cement available in both standard (EvoCem® 1) and low viscosity (EvoCem® 3) either with or without gentamycin antibiotic. Antibiotic cement is suitable for the treatment of revision arthroplasties, for septic replacements and for patients with infectious risks.

Choice of viscosity depending on the cement application mode and site of implantation:

- standard viscosity: digital application on superficial surfaces of the bone,
- low viscosity: syringe application in deep bone cavities or diaphyseal bones in association with a cement restrictor.

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
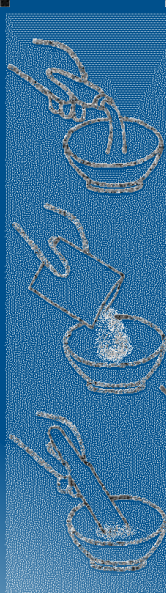
Mixing
Mélange

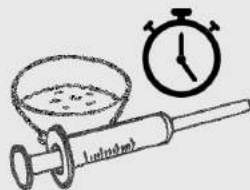Waiting
Repos

Digital application
Application manuelle

Syringe application
Application à la seringue

Application
Travail

CHARACTERISTICS

- Fast curing acrylic bone cement (9 to 12mn depending on version)
- Short mixing and waiting phases allow for longer working time
- Low powder polymer to liquid monomer ratio : reduces vapor diffusion and exothermic reaction
- Radiopaque agent: Barium sulphate

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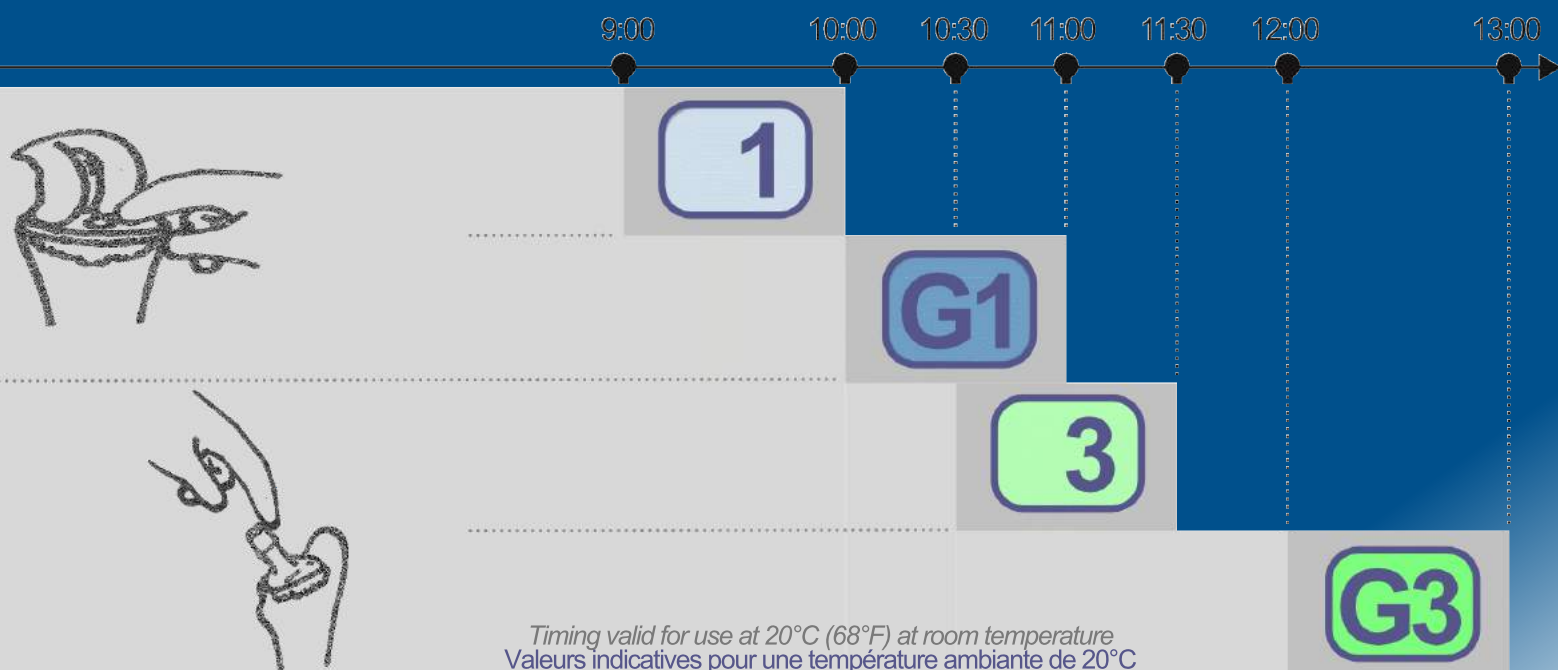
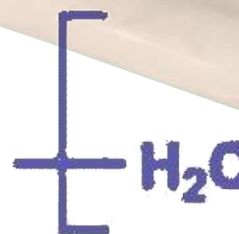
G1

G3

- Local and immediate antibiotic release into the prosthetic bone site
- Indications are limited to situations at risk of infection, removal of implants due to infection, and revisions
- Gentamycin antibiotic (aminosides) offer a large spectrum of antibiotic protection against the most commonly occurring bacteria

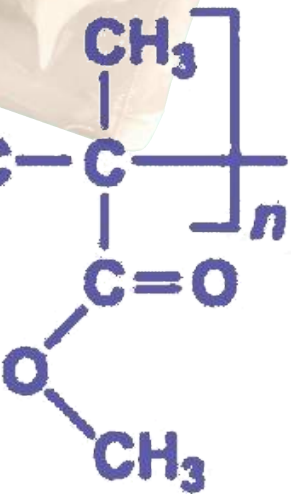
Specific formulation:

- flash release within the first hours of implantation (0.8mg as early as 18h after implantation)
- continuous efficient diffusion on a long term basis
- each individual 40g dose contains the equivalent of 1g of gentamycin base (gentamycin sulphate) and results in a local concentration exceeding the MIC (minimal inhibitory concentration)

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Preparation Mode

- 1- Make sure to use a clean and dry bowl, and an undamaged or worn spatula
- 2- Open the powder pouch with a pair of scissors
- 3- Grip the plastic tip and break the glass ampoule at its predetermined breaking point
- 4- Pour the liquid into the bowl
- 5- Immediately pour the powder over the liquid
- 6- Mix gently to avoid introduction of air bubbles, for 45"
- 7- Allow the cement to settle for 1'30"
- 7bis- For EvoCem® 3 and G3, immediately pour into the syringe and position the piston in the syringe body
- 7ter- Push the piston until the cement reaches the tip of the syringe, expelling the air
- 8- Once the cement no longer "sticks" to the gloves, spread/inject the cement on/into the bone or on the implant
- 9- Introduce the prosthesis
- 10- Impact the prosthesis with firm pressure
- 11- Remove the excess of cement with a curette
- 12- Finalize pressurisation of the implant in the cement
- 13- Remove excess cement and carefully clean around the implant
- 14- Once the cement begins to cure, further manipulation of the cement is no longer possible
- 15- Wait until complete curing of the cement as outlined in the time chart

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Notes

Mixing bowls and spatulas are available non sterile in packs of 10 units. These will require to be sterilized by autoclave at a t° of 134°C (273°F) during 18 minutes.

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Recommendations of use

- use with the usual precautions for patients with cardio-pulmonary risks
- thoroughly clean and prepare the implantation site with pulse lavage
- when used in a diaphysis:
 - use a cement restrictor (plug)
 - inject the cement in a retrograde fashion with a drain in place

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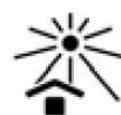
Recommendations for storage

- sterile products ready for use, pre-dosed
- do not use unless the packaging is intact
- single use, do not resterilise
- store in a clean dry room out of direct light
- do not store above 25°C (77°F)

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Material / Matériaux

Glass Ampoule: monomer (methyl-methacrylate, butyl-methacrylate, N-N dimethyl-p-toluidine and hydroquinone) sterilised by ultra-filtration and ethylene oxide after packaging.

Pouch: polymer (polymethyl-methacrylate, benzoyl and barium sulphate, gentamycin sulphate for the G versions) gamma ray sterilised (25 KGy) in double pouch packaging.

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